

Product Number ALCV-G14033-050
Lot # 1001007 (Contents; Samples: 1001007-1, -2, -3)
Expiration Date: **01 2011**
3 Levels x 3 vials x 0.5mL

Intended Use

NOD® Verification Samples are human Liquid assayed serum samples used to accomplish analyzer validation according to CLIA guidelines. The laboratory testing procedures are listed in the following text. Verification Samples must be run at least every six months for non-waived labs.

Summary and Principles

The use of independent quality verification sample materials is indicated as an objective assessment of the precision of methods and techniques in use and is an integral part of good laboratory practices. Three levels of verification samples are available to allow performance monitoring within the clinical range.

Reagent Composition

This product is prepared from purified human serum to which biochemical material (human and animal tissue extracts); drugs, chemicals, stabilizers and preservatives have been added. The verification samples are in a prepackaged liquid form to avoid potential error or contaminate being introduced during reconstitution.

Storage and Stability To achieve maximum shelf life for the Verification kit store unopened at $\leq -20^{\circ}\text{C}$. Store vials away from the light. Bilirubin may decrease over the product shelf life.

Thawed and Unopened: The Verification Samples can be used for up to **14 days** when stored unopened $2-8^{\circ}\text{C}$. For optimum Bilirubin and CO_2 stability avoid prolonged exposure of the Verification Sample vials to ambient air / room temperature / light.

Procedure

The verification samples should be treated the same as a patient sample and run according to the instructions accompanying the instrument, kit, or reagent being used. Before sampling the verification samples should be mixed thoroughly but gently.

Thaw **NOD®** Verification Samples at room temperature ($18-25^{\circ}\text{C}$) for 30 minutes or until completely thawed. Mix the vial thoroughly by inverting several times, before sampling gently swirls until homogeneous with no visible signs of precipitate. Avoid vigorous shaking. After sampling the Verification samples should be promptly re-capped and stored a $2-8^{\circ}\text{C}$. Dispose of at the end of day or upon completion of data collection.

Limitation of Procedure

(a) This product should not be used past the expiration date (b) if there is evidence of microbial contamination in the verification samples or excessive turbidity discard the vial (c) This Product is not intended for use as a standard.

The assay values recovered in the laboratory are method dependent and reflect reagent, method and technique and instrument variations. If methods and / or reagents are changed or modified the resulting assay value may be different.

Assignment of Values

The mean values printed on the circular were derived from replicate analyses on the Piccolo Clinical Chemistry Analyzer and are specific for this lot of Liquid Assayed Chemistry Verification Samples.

Specific Performance Characteristics

To ensure the reliability and usefulness of the verification samples, the product must be properly handled and stored as described.

Individual donor units used in the preparation of this product have been tested by FDA approved methods for anti-HIV 1 & 2, HBsAg, anti-HCV, HIV-1 antigens and Syphilis and found non-reactive. No test method can offer complete assurance that products derived from human source material will not transmit infectious diseases. Therefore, this product should be considered potentially infective and be treated in the same manner as a patient specimen.

Re-Ordering Information: Verification P/N ALCV-G14033-050 or Control P/N ALPC-G14026-100
By Ordering On Line at **NOVA-ONE.NET**; or Fax **818-348-9696**

NOVA-ONE@sbcglobal.net 818-348-1543 Toll Free 800-810-7488 Fax 818-348-9696
(REV 04-02-2010)



Assigned Mean Values (Representative Values)

Lot # 1001007 (Contents; Samples: 1001007-1, -2, -3)
Expiration Date: **01 2011**

METHOD: Abaxis Piccolo	Sample 1	Sample 2	Sample 3
Analyte, Units	Mean	Mean	Mean
ALT/SGPT U/L	20	756	1488
Albumin g/dL	1.9	4.2	6.2
Alkaline Phosphatase (ALP) U/L	28	1151	2076
Amylase, Pancreatic U/L	22	1252	2354
Aspartate Aminotransferase (AST/SGOT) U/L	21	810	1558
Bilirubin – Direct mg/dL	0.13	2.2	3.8
Bilirubin – Total mg/dL	0.4	2.7	4.7
BUN (Urea Nitrogen) mg/dL	5	71	153
Calcium Total mg/dL	4.2	9.6	14.2
Carbon Dioxide (CO ₂) mEq/L	34	21	11
Chloride mEq/L	84	105	126
HDL Cholesterol mg/dL	19	45	83
Total Cholesterol mg/dL	77	228	381
Creatine Kinase (CK) U/L	42	1803	3550
Creatinine mg/dL	0.33	7.83	15.59
GGT U/L	13	1381	2537
Glucose mg/dL	29	340	659
Lactate Dehydrogenase (LDH) U/L	68	382	754
Magnesium mg/dL	0.5	3.4	6.5
Phosphorus mg/dL	15	8.0	1.3
Potassium mEq/L	2.0	4.5	7.3
Protein, Total g/dL	3.5	7.0	10.9
Sodium mEq/L	114	138	167
Triglycerides mg/dL	45	193	345
Uric Acid mg/dL	1.2	6.8	12.7

Re-Ordering Information: Verification P/N ALCV-G14033-050 or Control P/N ALPC-G14026-100
By Ordering On Line at **NOVA-ONE.NET**; or Fax **818-348-9696**

NOVA-ONE@sbcglobal.net 818-348-1543 Toll Free 800-810-7488 Fax 818-348-9696
(REV 04-02-2010)



NOVA-ONE Diagnostics, LLC 22287 Mulholland Hwy, Calabasas, CA 91364



EMERGO EUROPE
Middelweg 15
2515 BR, The Hague
The Netherlands
Tel: +31 (0)70 345 8571
Fax: +31 (0)70 346 7291